

GD/FW2

Maternity Examination Model



Features:

In order to guarantee health of infant and its health in future, antenatal examination is very important. This model is designed basing on antenatal examination process. The skin is made of importing thermoplastic elastome and using technique of plastic injection mould. With features of lifelike shape, durability, realistic operation and reasonable structure, the model will give you realistic clinic feeling when operating. As a gynecologic teaching aid, it is perfect.

Functions:

1. Four maneuvers of Leopold can be actualized on the model. In order to gain realistic feeling, you can use the green bulb to pump air into the uterus. Put bubble cushion in the pelvic cavity, behind the symphysis ossium pubis. And cervices (foetus head) is uplifted, which indicate uterus hasn't entered into the pelvis. If bubble cushion taken out, cervices will enter into the pelvic cavity, indicating uterus has entered into the pelvis.
2. Embryo heart sound auscultation: controlled by computer chip. Students can hear embryo heart sound through stethoscope or loudspeaker.
3. The volume adjuster: adjust the volume. When the adjuster switch is at the middle, the loudspeaker hasn't sound and the stethoscope must be used. The heart rate adjuster: adjust the embryo heart rate (80, 120, 180/min).
4. Pelvic measurement
5. Mammary nursing

Apply Range:

1. Medical college, nursing school, gynecologic clinic: Instructing and operating of maternity examine
2. Clinic workers and nursing workers: Practice for vocational education
3. Primary sanitation organization: Popularizing medical technology

Maintenance and repair:

1. After training, clean the model.
2. If the model won't be used, clean the model. Pack and put it at shady, dry place. These methods will prolong its using life.
3. We guarantee to keep the model in good repair basing on the invoice and the copy of warranty card. The time limit of the free repair is one year.

Shanghai Honglian Medical Technology Group Co.,Ltd.

Marketing center: 5th Floor, No.1Bldg, Aijia International Bldg, 288#
Wuhua Rd., Shanghai China

Tel.: 0086-21-65223703

Fax: 0086-21-65223810

Zip code: 200086

Production base: No.88 Shande Road, Shanyang Town, Jinshan District,
Shanghai, China

Tel.: 0086-21-69595130 69595118

Fax: 0086-21-69595686

Zip code: 201508

Website : <http://en.honglian8.com> or www.generaldoctor.com

E-mail: sales@honglian8.com;

Honglian@honglian8.com;

Hong-lian@honglian8.com;